STUDENTS LIVING IN RESIDENCE LIFE, BEFORE LEAVING FOR THE SEMESTER

## PICK UP A COVID-19 TAKE HOME TEST KIT

AVAILABLE AT THE CENTRAL OFFICE.

\*BRING YOUR POLAR EXPRESS CARD

UA is an AA/EO employer and educational institution and prohibits illegal discrimination against any individual: www.alaska.edu/nondiscrimination.

## WHAT'S IN THE



## HOW TO TAKE THE TEST \*TAKE 72 HOURS BEFORE RETURNING TO CAMPUS

Create an account at everlywell.com/register and register your test kit.

Watch the collection video at everlywell.com/collect.

Fill out the kit ID sticker.
Write your full name, date of birth, and date of collection.
Make sure the sticker information matches your register information.

Collect the sample by inserting the swab 1 inch in your nostril and rotating it three times.

Assemble your shipping box, disinfect the mailer, and ship your sample.

Monitor your kit status through email and text. Test results available 24-48 hours of the lab receiving your sample.

<sup>\*</sup>Detailed instructions provided with the kit.