

# WHAT'S IN THE KIT



# HOW TO TAKE THE TEST

**\*TAKE 72 HOURS BEFORE RETURNING TO CAMPUS.**

- 1 Create an account at [everlywell.com/register](https://everlywell.com/register) and register your test kit.
- 2 Watch the collection video at [everlywell.com/collect](https://everlywell.com/collect).
- 3 Fill out the kit ID sticker. Write your full name, date of birth, and date of collection. Make sure the sticker information matches your register information.
- 4 Collect the sample by inserting the swab 1 inch in your nostril and rotating it three times.
- 5 Assemble your shipping box, disinfect the mailer, and ship your sample.
- 6 Monitor your kit status through email and text. Test results available 24-48 hours of the lab receiving your sample.

*\*Detailed instructions provided with the kit.*